


# Gas Outlets - Diamond I

Main Back

Parts Breakout

6803-7200-002 Primary Check Repair Kit, Includes 1 each of items: 4, 5, 7, 8, 9, 18

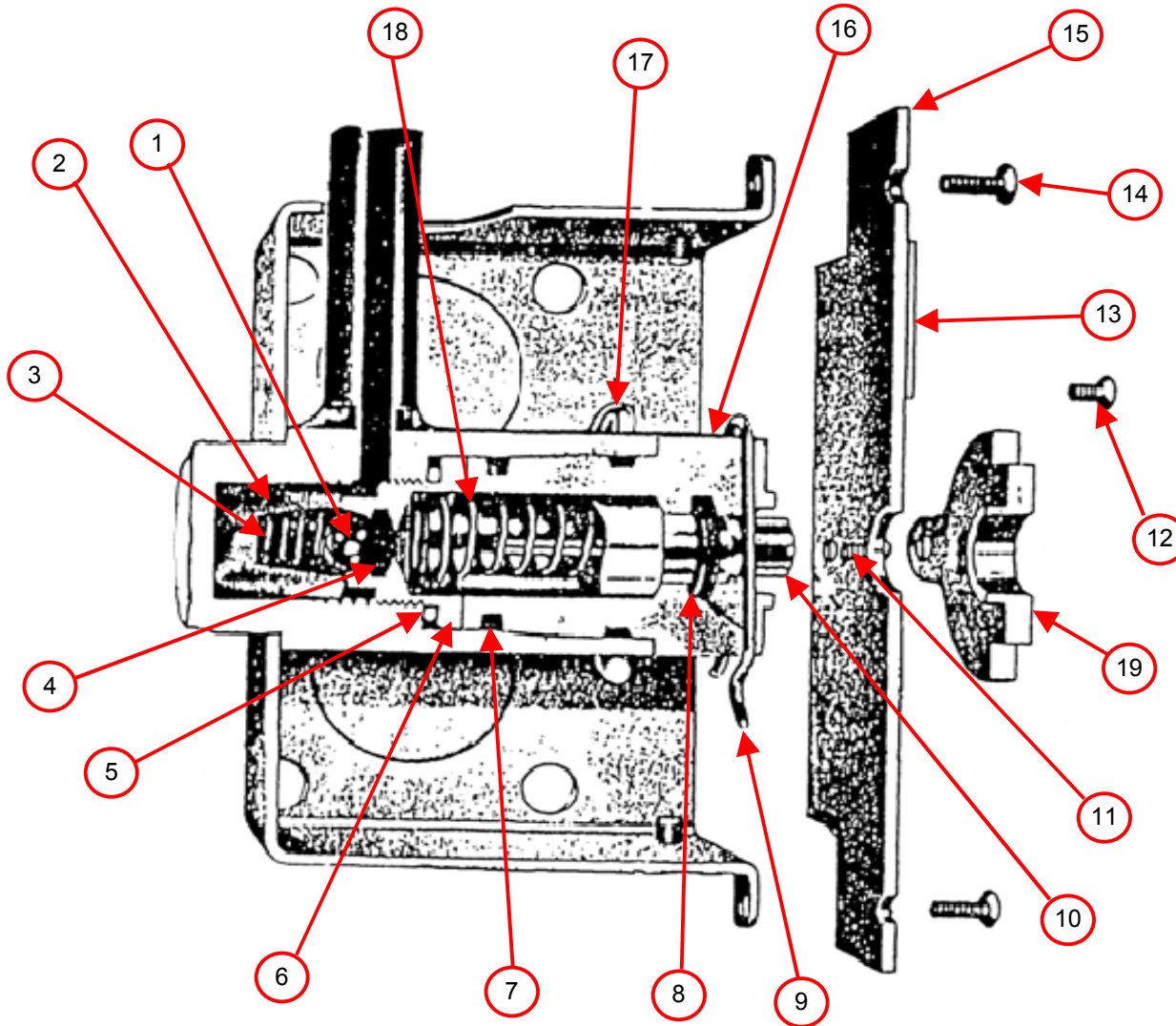

Item #	Part Number	Part Description
1*	0409-1697-300	Secondary Check Ball
2*	0214-7162-300	Inlet Screen
3*	0203-3230-300	Secondary Check Spring
4*	0210-0629-300	Secondary Check O-Ring
	0210-0492-870	Secondary Check O-Ring, Pkg of 10
5	0210-0703-300	O-Ring
	0210-0497-870	O-Ring, Pkg of 10
6	0221-2524-500	Retainer, Pressure
	0175-0514-000	Secondary Check Removal Tool
7	0210-4512-300	Body Quad Seal Ring
	0210-4513-870	Body Quad Seal Ring, Pkg of 10
8	0210-0572-300	Primary Check O-Ring
	0210-0489-870	Primary Check O-Ring, Pkg of 10
9	0203-5064-300	Locking Spring
10	6803-7030-030	Piston
11	0140-6113-105	Screw, 3-48 x 5/16"
	0140-6224-106	Screw, 6-32 x 3/8" (Disc w/Ears)
	0140-5213-107	Screw, 3-48x7/16" (Disc w/o Ears)
	0205-4610-300	Name Plate, O2
13	0205-4611-300	Name Plate, N2O
	0205-4612-300	Name Plate, VAC
	0205-4614-300	Name Plate, AIR (Yellow)
14	0142-2243-210	Screw, #6x5/8" Name Plate Bracket
15	0221-2515-542	Name Plate Bracket, Flat
16	0221-2420-500	Primary Check Body, New Style
17	0203-5081-304	Spring Clip
18	0203-4042-300	Primary Check Spring
19	0221-2400-100	Keying Disc With Ears, O2
	0221-2401-100	Keying Disc With Ears, N2O
	0221-2402-100	Keying Disc With Ears, VAC
	0221-2422-100	Keying Disc With Ears, AIR
	0221-2405-100	Keying Disc Without Ears, O2
	0221-2406-100	Keying Disc Without Ears, N2O
	0221-2407-100	Keying Disc Without Ears, VAC
	0221-2421-100	Keying Disc Without Ears, AIR

**Note:** If you have metal keying disc, you will have to order new primary check body and new keying disc.  
Tool for removing secondary check retaining ring: 0175-0514-000.

\* Not used on vacuum outlet  
**Andersen Medical Gas**  
 12 Place Lafitte  
 Madisonville, LA 70447  
<https://www.TheMedicalGas.com>  
 1-866-288-3783