


QUICK-CONNECT RECESSED OUTLET – AQ11XX SERIES

ALLIED/CHEMETRON Compatible

Oxygen Outlet Shown

Andersen Medical Gas
12 Place Lafitte
Madisonville, LA 70447
<https://www.TheMedicalGas.com>
1-866-288-3783

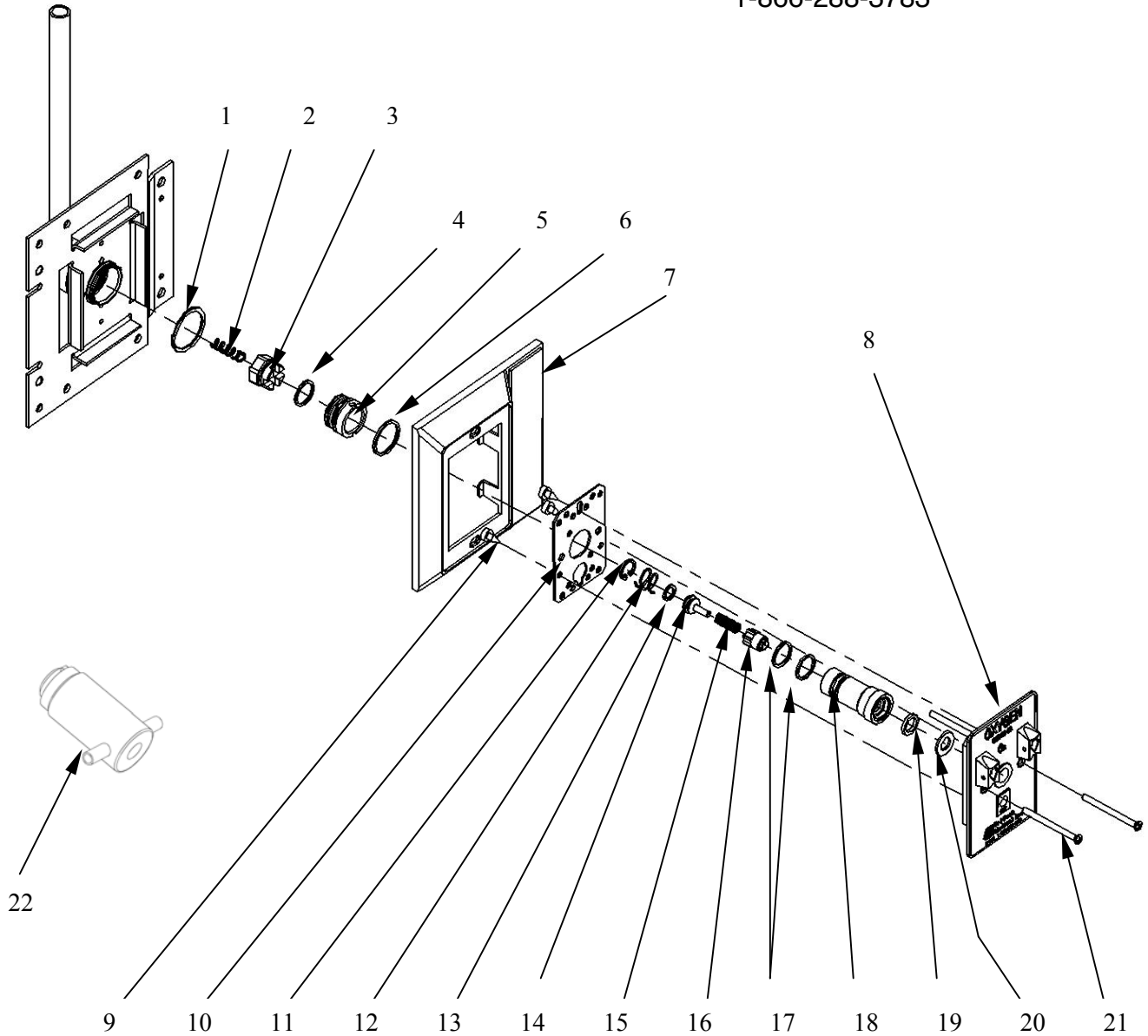

Item 8 Includes Items 9 through 21 in the Finishing Kit

Quick-Connect Recessed Outlet – AQ11XX SERIES Bill of Material

Item	Part Number	Description	Quantity	Note
1.	12-1038R	Retaining Ring	1	
2.	13-1003R	Spring	1	
3.	AQ1000-4R	QC Poppet	1	
4.	11-0008R	Poppet O-Ring	1	A
5.	Q1000-3R	Check Retainer Bushing	1	
6.	11-0006R	Bushing O-Ring	1	B
7.	AQ1000-9R	Name Plate Housing	1	
8.	See Below	AQ Finishing Kit Assemblies	See Below	
9.	12-3027R	Mounting Screws	3	
10.	AQ1000-12R	Indexing Housing Cover	1	
11.	12-1020R	Retaining Ring	1	
12.	13-1006R	Taper Spring	1	
13.	11-0016R	Poppet O-Ring	1	
14.	AQ1000-7R	Poppet	1	
15.	13-1007R	Poppet Spring	1	
16.	AQ1000-5R	Front Poppet	1	
17.	11-0008R	Body O-Ring	2	C
18.	AQ1000-2R	Front Body	1	
19.	11-0005R	Front Body O-Ring (Orange)	1	C
20.	12-1002R	Plated Washer	1	
21.	12-3025R	Mounting Screws	2	

Repair Tools:

22.	Q9900SD	Secondary Check Removal Tool	1	
-----	---------	------------------------------	---	--

Item 8-Outlet Finishing Kits (All kits are assembled with Items 9 through 21)

	AQ1004AR	Nitrous Oxide	1	
	AQ1016AR	Medical Air	1	
	AQ1022AR	Medical Vacuum	1	
	AQ1024AR	Oxygen	1	
	AQ1032AR	WAGD	1	

Primary Valve Kit (Kit Includes Items 11 through 18 Pre-Assembled Plus Items 19 and 20)

	AQ1100RC	Primary Valve Kit for all Gases	1	
--	----------	---------------------------------	---	--

Secondary Valve Kit (Kit Includes Items 1 through 5)

	AQ1000KR	Secondary Valve Kit for all Gases	1	
--	----------	-----------------------------------	---	--

Note: **A:** Install O-Rings items 4, on Poppet item 3.
 B: Install O-Ring item 6, on to Check Retaining Bushing item 5.
 C: Install O-Rings item 17 on to and insert O-Ring item 19 into Front Body item 18.